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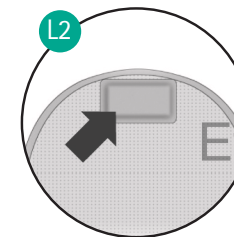
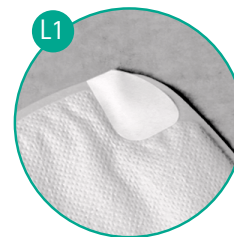
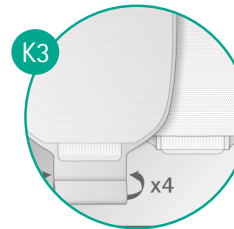
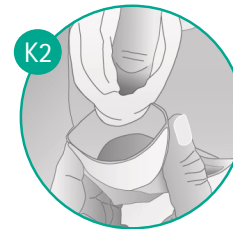
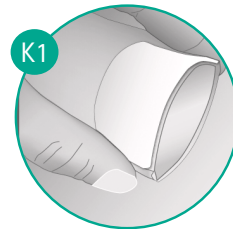
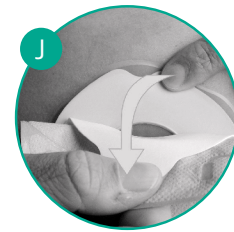
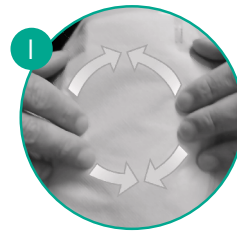
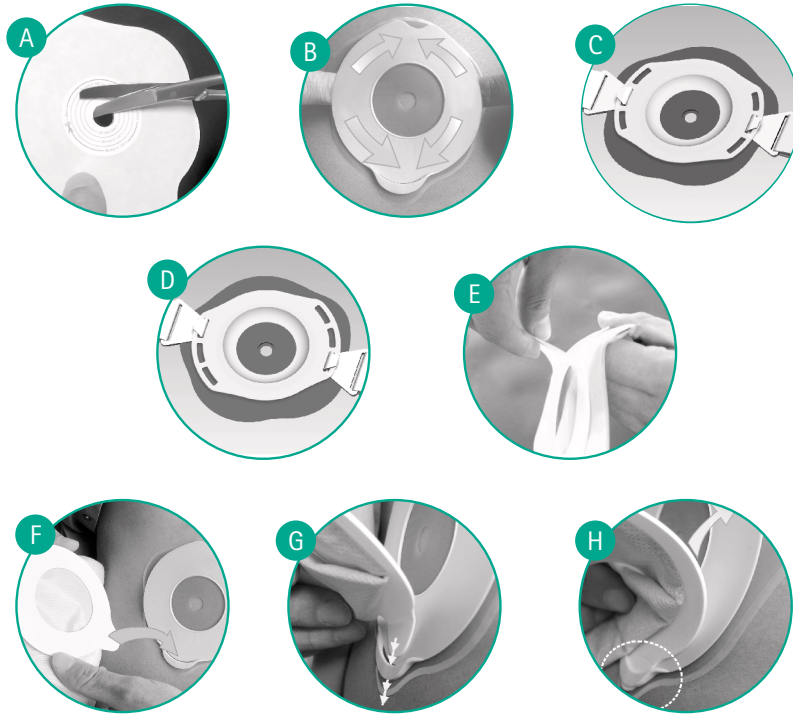
Flexima® / Softima® Key

EN Instructions for Use

What Flexima® / Softima® Key 2-Piece is and what is used for:

Flexima®- Softima® Key is a 2-Piece collection system for enterostomates or urostomates comprising of:

- A base plate with a skin protector and soft flange, also available* with soft convexity and loops to attach the B. Braun Belt.
- Different types of collecting pouches*:
 - Closed for colostomates,
 - Drainable for enterostomates with either "Roll'Up" outlet or a cap for Maxi Drainable allowing connection to a B. Braun distal bag.
 - Drainable for urostomates, with a drain tap.

The unique feature of the system is an integrated guiding system that allows easy positioning of the adhesive pouch onto the flexible base plate.

Flexima® / Softima® Key may be used concomitantly with B. Braun stoma care accessories.

Intended purpose and Indication:

The product is indicated for collection and containment of stools or urine by patients with enterostomy or urostomy. The product is not intended to be used on a skin or mucus membrane presenting a pathological change or change following disease or a wound.

Closed pouches are primarily intended for colostomates; drainable "Roll'Up" pouches for colostomates with soft or liquid stools and ileostomates; Maxi drainable pouches for ileostomates and jejunostomy (in case of high flow liquid stools) and uro pouches for urostomates.

Clinical benefits & performances: Ostomy appliances compensate for the loss of faecal and/or urinary continence.

Patient population: Enterostomates and urostomates adults or children above four years.

Contraindication:

For the closed pouch Flexima/ Softima Key, we do not recommend for use by colostomates with liquid stools.

Residual risks or side effects:

Peristomal skin complications can occur while using Flexima® / Softima® Key.

1 Preparing the skin

Wash your hands before starting the procedure. Gently clean the peristomal area with warm water. Avoid ether, alcohol or any other topical product which could interfere with base plate adhesion. Carefully dab dry the skin with soft and non-shedding cloths.

2 Preparing the base plate

Separate the cardboard from the base plate. If you are using a Cut-To-Fit product, cut the central opening of the hydrocolloid adhesive to the size of your stoma (A) possibly with the help of the Stoma Guide. Do not cut beyond the maximum possible diameter and make sure that there is always a 2 to 3 mm margin around your stoma in order not to strangle or hurt the stomal mucosa. Note: Pre-Cut products are not intended to be cut out.

3 Applying the base plate

Peel off the release. Apply the base plate around your stoma. In order to improve the adhesion, place hands over the base plate to warm it up.(B)

4 Applying the pouch (closed & drainable)

For drainable "Roll'Up" and "High Flow" pouches, close the closure outlet:

- Drainable "Roll'Up" pouches: Fold the extremity of the sleeve four times using the guides and fold the tab back. To seal the outlet, make sure that the two self-fastening strips are aligned, facing each other and apply pressure on all their length (K3).

MD

- EN Medical device as per 2017-745/EU
- DE Medizinprodukt gemäß 2017-745/EU
- ES Producto sanitario según el Reglamento (UE) 2017-745
- FR Dispositif médical selon 2017-745/EU
- IT Dispositivo medico secondo 2017-745/UE
- NL Medisch hulpmiddel conform 2017-745/EU
- BG Медицинско изделие съгласно 2017-745/EU
- CS Zdravotnický prostředek dle nařízení (EU) 2017/745
- DA Medicinsk udstyr i henhold til 2017-745/EU
- EL Ιατροτεχνολογικό προϊόν σύμφωνα με τον Κανονισμό 2017/745/EE
- ET Meditsiiniseade 2017-745/EU kohaselt
- HU Orvosi eszköz, a 2017/745/EU rendeletnek megfelelően
- JA 2017-745/EUに從つた医療機器
- LV Medicīniskā ierīce saskaņā ar 2017-745/ES
- MS Peranti perubatan berdasarkan 2017-745/EU
- NO Peranti perubatan berdasarkan 2017-745/EU
- PL Wyrób medyczny zgodnie z rozporządzeniem (UE) 2017/745
- PT Dispositivo médico de acordo com a diretiva 2017-745/EU
- RO Dispozitiv medical conform 2017-745/EU
- RU Медицинское изделие согласно 2017-745/EU
- SK Zdravotnícka pomôcka podľa normy 2017-745/EU
- SL Medicinski pripomoček v skladu z 2017-745/EU
- SR Medicinsko sredstvo u skladu sa 2017-745/EU
- SV Medicinteknisk produkt enligt 2017-745/EU

Effective

Effective

- Drainable pouch with cap, Maxi Drainable: Close the cap as far as maximum.

Separate the pouch walls by crumpling it slightly to introduce air into it.

Remove the release paper that protects the adhesive of the pouch (E).

Grasp the pouch by the sides.

Insert the tab of the pouch into the slot of the base plate (F) and (G). Avoid placing your fingers on the adhesive.

Adjust and make the pouch adhere onto the base plate without making folds (H).

Apply the adhesive carefully by exerting a pressure onto the stuck area (I).

Make sure the pouch is well secured by pulling on it slightly.

5 Emptying of drainable pouches

Position yourself appropriately.

- Drainable "Roll-Up" pouches:

- Release the two self-fastening strips and unroll the sleeve. Be careful, emptying will start right at this point. To fully open the outlet, apply slight pressure with your fingers on guides (K1).

- Once the emptying is complete, expel, as much as possible, any matter still in the sleeve by pressing it lightly from top to bottom. Carefully clean its extremity avoiding the use of shedding cloths (K2).

- Then close the outlet again.

- Please note: the closed sleeve is hidden under the soft backing material (K3).

- Maxi Drainable pouches with cap:

Remove the cap and point it towards the toilet bowl. Once the emptying is complete, clean the cap.

After emptying of drainable pouches, close the closure outlet again as indicated above.

6 Using B. Braun belt with Flexima/Softima Key convex

If you wish to use a B. Braun belt, choose two loops that are diametrically opposed (C,D).

7 Using filter protection stickers

Before bathing or showering, stick the filter protection sticker over the gas evacuation openings (L1,L2) in order to maintain filter performances. For Split cover, this must require to lift the comfort film and to stick the filter protection filter directly on the pouch films. After bath or shower, remove the filter protection stickers.

8 Removing the products

- Removing the pouch:

Maintain the base plate by the prehension tab and unstick the pouch progressively with the other hand by pulling on the tab of the pouch. For the Urostomy pouch*, unstick the pouch either with the thumb inserted into the specific cut-out, or with the specific accessory inserted in the slot (J). Turn to unstick the pouch. Complete the un-sticking progressively.

- Removing the base plate:

Pull slightly on the top of the base plate and remove the base plate from the top to the bottom, while maintaining gentle pressure on the skin around the stoma with your other hand.

* Availability may vary from one country to another.

If you note any change on your stoma and/or peristomal skin, please contact your health care professional.

National regulations for the destruction of potentially infectious materials must be observed.

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