Stabimed® fresh

EN Solution for instrument disinfection of dental and surgical instruments and invasive medical devices prior to sterilization.

Information for use: Manual disinfection and cleaning (including ultrasound cleaning) of surgical instruments, rigid and flexible endoscopes, laboratory and anaesthesia equipment. After soaking in ready-to-use Stabimed® fresh working solution, rinse the instruments thoroughly under cold running tap water. Perform the final rinse in demineralized or distilled water. Dry instruments with an absorbent, lint-free cloth towel.

Prior to the first use of Stabimed® fresh the instrument bath shall be cleaned with water and e.g. Helizyme, to remove potential residues of previously used products.

Information: Do not use in combination with aldehyde-based disinfectants! Limited compatibility with silicone-based products. Stabimed fresh is not full virucidal against non-enveloped viruses, therefore it is not suitable for the terminal disinfection of heat sensitive semicritical and critical devices.

Intended user: Professionals in hospitals, medical practice and Sterile Services Units, physicians and dentists

Notice to the user: Any serious incident that has occurred in relation to the device should reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established. For professional use. Date of issue of the instruction for use: 2021-04-26. Date of the last revision: 2021-04-26.

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