Prontosan® Debridement Pad

EN Wound debridement pad

Directions for use

Prontosan® Debridement Pad consists of microfibres that debride and an absorbent backing layer. The device is individually packed and sterilised with ethylene oxide. Prontosan® Debridement Pad is intended for single use only and must not be resterilised. The device is sterile as long as the pack remains unopened and undamaged. The integrity of the sterile barrier must be checked visually before use. In case of damages (punctures, holes, openings, or other damages) compromising the packaging integrity, the device is to be discarded.

Indications

Prontosan® Debridement Pad is used for rapid, effective and safe debridement.

Prontosan® Debridement Pad frees the wound from coatings and dead cell residues (debris) and absorbs excess exudates. Intact tissue is spared.

Prontosan® Debridement Pad produces good results even with scaly and necrotic coatings, if they are subject to prior or autolytic treatment.

Composition: polypropylene, polyester

Instructions for use

- 1) Remove the old wound dressing.
 - 2) Perform debridement:
 - a) The unmarked side of the pad is moistened with Prontosan® wound irrigation solution or, if not available, with saline solution or distilled water. If local guidelines exist, these must be observed.
 - b) Applying gentle pressure, wipe the moistened side over the wound and/or adjacent skin in a circular motion. Following debridement, it is advisable to irrigate the wound thoroughly for a second time with Prontosan® wound irrigation solution and to continue with standard treatment.
- Precautions**
- Do not use Prontosan® Debridement Pad as a wound dressing.
 - The device must not be used in case of known intolerance or an allergy to one or more of its constituents.
 - Prontosan® Debridement Pad must not be cut to size.

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